

Air Bubble Call Pendant

The Advance Care Air Bubble Call Pendant has been designed with the needs of 'restricted movement' residents in mind - for example those with arthritis. Only the slightest pressure is required from the resident to activate the device, thanks to the cleverly designed pneumatic activation. The device is easy to hold as it is very light and has a clip for easy attachment to resident's bed or clothing to keep them within easy reach.

As the Air Bubble Call Pendant is plugged into the Nurse Call Button via the pendant socket on the front of the plate, the Call Device essentially acts as an extension of the Nurse Call Button, allowing for the easy access of a Call Point without having to physically touch the wall.

Although an extension of the Nurse Call Button, the Air Bubble Call Pendant has its own easily identifiable alert, such as 'Pendant Room 1', when it is activated.

When a call is made, a high visibility reassurance light illuminates the 'CANCEL' button on the Call Button plate and the Call Button may also 'beep' periodically (if activated), providing reassurance to the user.

The Nurse Call alert is also immediately displayed at the Nurse Station and sent to associated paging devices.

The Advance Care IP Nurse Call System includes a 'Pendant Removed' alarm feature which generates an alert if the pendant is accidently pulled out or incorrectly plugged in, a beneficial feature with resident safety as key.

The pendant itself is totally waterproof, completely safe in oxygen rich environments and is UL1069 listed.

Available with an 8 or 12 foot cable, the Air Bubble Call Pendant integrates with most Nurse Call Systems and is a valued addition to any Advance Care IP Nurse Call installation.

## **Features**

- Suitable for residents with restricted movement
- Soft pneumatic activation
- Integrates with most Nurse Call Systems
- 6.35mm industry standard Pendant Plug
  - Self-configured alarm notification
  - · Safe in Oxygen rich environments; UL1069 rated
    - Bed clip as standard
    - Cord length options

Soft Pneumatic Activation

6.35mm Pendant Plug

Code	Description
AC-PA/8	Air Bubble Call Pendant (2400mm/8 foot cord length) - white or grey
AC-PA/12	Air Bubble Call Pendant (3700mm/12 foot cord length) - white or grey





## **Design Specifications**

Feature	Description
Height	25mm
Width	40mm
Depth	38mm
Weight	190 grams
Connectors	6.35mm Stereo Phono Plug
Temperature	-20°C to 50°C
Humidity Range	0 to 90%
Rating	UL1069

## Compliance

Designed to comply with:

- AS/NZS 3811 Hard Wired Patient Alarm Systems
- AS/NZS 2999 Alarm Systems for the Elderly and Other Persons at Risk
- AS/ACIF S009 Installation Requirements for Customer Cabling
- AS/NZS 3000 Wiring Rules
- AS/NZS CISPR22 (Class B) Electromagnetic Compatibility (EMC)

© 2022 Advance Care™

All rights are reserved.

Advance Care™ reserves the right to make changes in specifications and/ or discontinue any product at any time without notice or obligation and will not be liable for any consequences resulting from the use of this publication.

Authorised Distributor

