

The Advance Care Silicone Light Switch Call Pendant has been ergonomically designed, ensuring the device is easy for residents to hold and activate. The Silicone Light Switch Call Pendant also has a clip for easy attachment to resident's bed or clothing to keep it within easy reach.

Acting as both a standard 'Call Pendant' and a 'Light Activation' switch, the device must be partnered with the Light Switch Relay Unit in order to be operational. The 'Light Activation' switch can be configured to switch the resident's reading lamp on, or illuminate the entire room.

As the Silicone Light Switch Call Pendant is plugged into the Nurse Call Button via the tough, fully-moulded 6.35mm pendant socket on the front of the plate, the Call Pendant essentially acts as an extension of the Nurse Call Button, allowing for the easy access of a Call Point without having to strain reaching the wall.

Although an extension of the Nurse Call Button, when activated, the Silicone Light Switch Call Pendant has its own easily identifiable alert, such as 'Pendant Room 1'.

When a call is made, a high visibility reassurance light illuminates the 'CANCEL' button on the Call Button plate and the Call Button may also 'beep' periodically (if activated), providing reassurance to the user. The Nurse Call alert is also immediately displayed at the Nurse Station and sent to associated paging devices.

The Advance Care IP Nurse Call System includes a 'Pendant Removed' alarm feature which generates an alert if the pendant is accidently pulled out or incorrectly plugged in, a beneficial feature with resident safety as key.

The Silicone Call Pendant is constructed of high impact silicone and designed to withstand the stresses of being dropped on a regular basis.

This product integrates with most Nurse Call Systems and is a valued addition to any Advance Care IP Nurse Call installation.



Code	Description
AC-SPL8	Silicone Pendant with Light Switch 8ft
AC-SPL12	Silicone Pendant with Light Switch 12ft
AC-IPRLY	Light Relay including Housing





Design Specifications

Feature	Description
Height	87mm
Width	44mm
Depth	17mm
Weight	192 grams / 210 grams
Material	Silicone
Connectors	6.35mm Phono Plug
Temperature	-20°C to 50°C
Humidity Range	0 to 90%
IP Rating	IP64

Compliance

Designed to comply with:

- AS/NZS 3811 Hard Wired Patient Alarm Systems
- · AS/NZS 2999 Alarm Systems for the Elderly and Other Persons at Risk
- · AS/ACIF S009 Installation Requirements for Customer Cabling
- AS/NZS 3000 Wiring Rules
- AS/NZS CISPR22 (Class B) Electromagnetic Compatibility (EMC)

© 2022 Advance Care™

All rights are reserved.

Advance Care™ reserves the right to make changes in specifications and/ or discontinue any product at any time without notice or obligation and will not be liable for any consequences resulting from the use of this publication.

Authorised Distributor

